

QUICK REFERENCE GUIDE

PRE-SURGERY SCREW-RETAINED HYBRID DENTURE

DOCTOR

1ST APPOINTMENT

Take preliminary impressions of edentulous arch & opposing dentition

* Taking master impressions using "your" custom tray eliminates one appointment.

2ND APPOINTMENT

Take master impression using lab-fabricated custom tray.

3RD APPOINTMENT

Obtain jaw relation records, select denture tooth shade & mold.

4TH APPOINTMENT

Trial denture set-up try-in, check bite & provide photos.

- If immediate denture,

- Optional patient/doctor set-up evaluation OR final delivery on 4th appointment.

5TH APPOINTMENT

Final delivery of prosthesis & surgical guide for immediate load procedure.

FRIENDSHIP

1ST APPOINTMENT (STAGE 1)

Pour models, fabricate custom tray.

In Lab - 5 Days

2ND APPOINTMENT (STAGE 2)

Box & pour master model, fabricate bite blocks.

In Lab - 5 Days

3RD APPOINTMENT (STAGE 3)

Articulate models, set denture teeth in wax.

In Lab - 6 Days

4TH APPOINTMENT (STAGE 4)

Process denture & clear duplicate denture. Prepare for implant surgical guide/bone reduction guide.

In Lab - 10 Days

5TH APPOINTMENT (STAGE 5)

Schedule Chairside Denture Conversion.



INFINITY
LEVEL SPONSOR 2016-2021




A TRADITION OF EXCELLENCE

6826 Eastern Avenue • Baltimore, MD 21224

P: (443) 503-5301 • F: (410) 633-1498

E: rick@friendshipdentallab.com

WWW.FRIENDSHIPLABS.COM

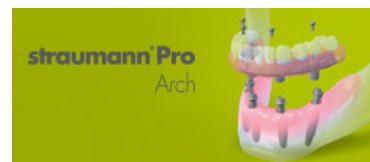

All-on-4™



RevitaliZe™

ZIMMER BIOMET

Your progress. Our promise.™

QUICK REFERENCE GUIDE

POST-SURGERY SCREW-RETAINED HYBRID DENTURE

DOCTOR

1ST APPOINTMENT

Implant/Abutment level open tray impressions.
Optional: Chairside Luting Together of impression transfers with open tray pick-up of splinted transfers.

2ND APPOINTMENT

Obtain jaw relation records, select denture tooth shade & mold. Verify passive fit of verification jig radiographically & send copy to Friendship for documentation.

3RD APPOINTMENT

Trial denture set-up try-in, check bite & photos.

4TH APPOINTMENT

Combination Ti Framework/set-up try-in.

5TH APPOINTMENT

Final delivery of prosthesis.

6TH APPOINTMENT

Post-delivery check of occlusion, re-torque screws if needed, optional night guard (*highly recommended to protect investment*). General hygiene check-up.

FRIENDSHIP

1ST APPOINTMENT (STAGE 1)

Pour soft tissue master analog model, fabricate screw-retained bite block & verification jig if impression transfers not luted together.
In Lab - 6 Days

2ND APPOINTMENT (STAGE 2)

Articulate models, set denture teeth in wax
In Lab - 6 Days

3RD APPOINTMENT (STAGE 3)

Fabricate Titanium framework, transfer set-up to Ti framework.
In Lab - 15-20 Days

4TH APPOINTMENT (STAGE 4)

Final adjustments & process hybrid to opaqued Ti. framework.
In Lab - 10 Days

5TH APPOINTMENT (STAGE 5)

Optional Night Guard
Pour models, articulate & fabricate nightguard
In Lab - 6 Days



INFINITY
LEVEL SPONSOR 2016-2021




A TRADITION OF EXCELLENCE

6826 Eastern Avenue • Baltimore, MD 21224

P: (443) 503-5301 • F: (410) 633-1498

E: rick@friendshipdentallab.com

WWW.FRIENDSHIPLABS.COM

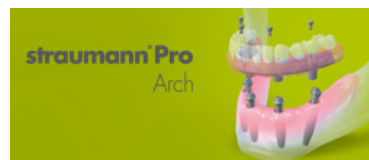

Revitalize™

ZIMMER BIOMET
Your progress. Our promise.™